


# Heelift® Glide Suspension Boot

## Application and Fitting Guide

- Choice of interior (smooth or convoluted)
- Available in Standard, Bariatric, and Petite
- Made of latex-free polyurethane foam.
- Recommended for single-patient use.

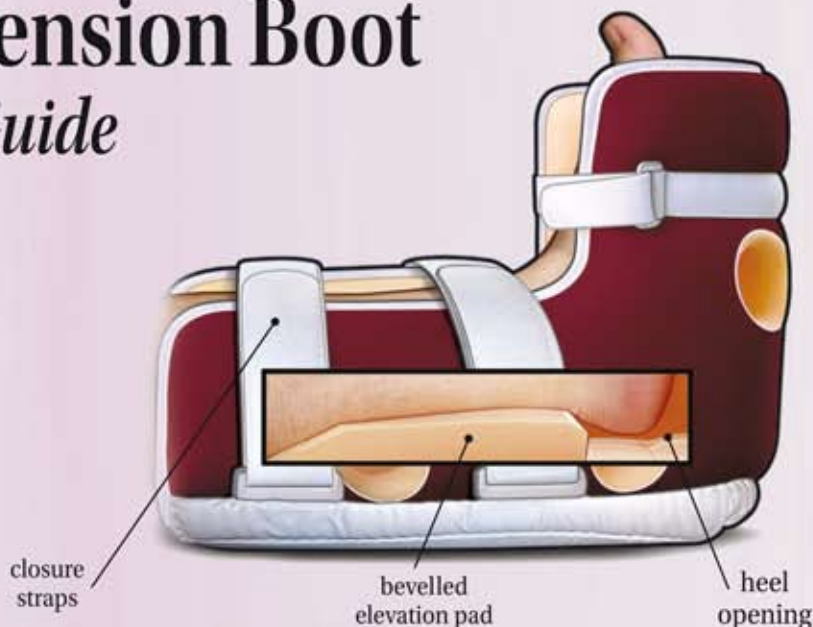

### Easy Application and Fit

Available in multiple languages:

<http://www.heelift.com/heelift-instructions.html>



1 Place foot inside boot with the heel above the opening.

2 Thread straps through D-rings. Secure the straps.

3 Test the fit under straps. **APPLY STRAPS LOOSELY.**

4 Test the fit under heel opening. Boot should not move on the leg.

### Proper Sizing and Care



**Petite**  
7.75" H x 10.5" L

**Standard**  
10" H x 12" L

**Bariatric**  
11.75" H x 15" L

<b>Standard</b>	120 - 250 lbs (54 - 113 kg)	8 - 14 in (20 - 36 cm)	60 - 77 in (152 - 196 cm)
<b>Bariatric</b>	220 - 600 lbs (100 - 271 kg)	12 - 23 in (30 - 58 cm)	65 - 80 in (165 - 203 cm)
<b>Petite</b>	70 - 130 lbs (32 - 50 kg)	6 - 10 in (15 - 25 cm)	46 - 64 in (117 - 163 cm)

Heelift® Glide Patent Nos. - 5,449,339 & 7,458,948  
Patents Pending  
Heelift® Glide Suggested Code - E0191



# Optional Customisation

Visit us online for more details:  
<http://www.heelift.com/customize.html>

Boot Rotation



Hip Rotation



SCD with Heelift®



Achilles Tendon



Foot Drop



Malleolar Decubitus



## Innovators In Pressure Ulcer Solutions

Heelift® AFO Boot



[www.heeliftafo.com](http://www.heeliftafo.com)

Heelift® Traction Boot



[www.tractionboot.com](http://www.tractionboot.com)

Elbowlift® Suspension Pad



[www.elbowlift.com](http://www.elbowlift.com)

Heelift® Suspension Boot



[www.heelift.com](http://www.heelift.com)



**V-M ORTHOTICS LIMITED**  
 St Peter's Church, Ubbeston, Halesworth, Suffolk IP19 0EX  
 TEL: 01986 798120 FAX: 01986 798040  
[www.vmorthotics.co.uk](http://www.vmorthotics.co.uk) email: [info@vmorthotics.co.uk](mailto:info@vmorthotics.co.uk)



© 2012, DM Systems, Inc. All rights reserved.